# 技术规格/Technical Data Sheet

E-40R HF XD55A L

## MONOBLOCK E-40R HF XD55A L

编码 / CODE: **30.04.003.001**

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WHEN NOT USED, AFTER 3 MONTH PERIOD, PROCEED TO INSERT RESTORATION FOLLOWING THIS CHART:

**1- FLUORO: FROM 60 kV to 120kV(step 10kV), 2 mA, 30 SEC ON + 10 SEC OFF; PAUSE 5 MIN**

**2- 90 kV, 4 mA, 60 sec. ON + 5 min. OFF; 120 kV, 4 mA, 60 SEC ON + 5 min OFF**

**IF INSTABILITIES OCCUR, PLEASE STOP THE PROCESS, WAIT 5 MINUTES AND RESUME THE PROCEDURE FROM THE BEGINNING.**

**CE** 0051

IMD CHINA CO., LTD.

1 -2/F, BUILDING 12, No. 28 YUHUA RD. AREA B, TIANZHU AIRPORT INDUSTRIAL ZONE, BEIJING, CHINA

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#### X射线管组件性能 / PERFORMANCE OF MONOBLOCK

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### 典型电气参数 / TYPICAL ELECTRICAL CHARACTERISTICS

|                                  |             |
|----------------------------------|-------------|
| 主绕组到公共端 / PRINCIPAL TO COMMON    | 25 $\Omega$ |
| 副绕组到公共端 / AUXILIARY TO COMMON    | 50 $\Omega$ |
| 主绕组到副绕组 / PRINCIPAL TO AUXILIARY | 75 $\Omega$ |

## 高压变压器电源 / H.V. TRANSFORMER POWER SUPPLY

### IMD逆变器E-9040-5 HF/ IMD INVERTER E-9040-5 HF

|                                                                      |        |
|----------------------------------------------------------------------|--------|
| 工作频率 / OPERATING FREQUENCY                                           | 20 kHz |
| 最高峰值输入电压 (X和Y之间)<br>MAXIMUM INPUT PEAK VOLTAGE (X AND Y CONNECTIONS) | 350 V  |
| 最大峰值输入电流 (约为)<br>MAXIMUM INPUT PEAK CURRENT (APPROX.)                | 130 A  |

## 输出信号 / OUTPUT SIGNALS

|                                    |                 |
|------------------------------------|-----------------|
| kV+ (模拟信号 / ANALOG SIGNAL)         | 0 $\div$ 6,0 V  |
| kV- (模拟信号 / ANALOG SIGNAL)         | 0 $\div$ 6,0 V  |
| mA+ AND mA- (模拟信号 / ANALOG SIGNAL) | 0 $\div$ 100 mA |

## 热保护 - 常闭 - (逻辑信号) THERMAL SECURITY - NORMALLY CLOSED - (LOGICAL SIGNAL)

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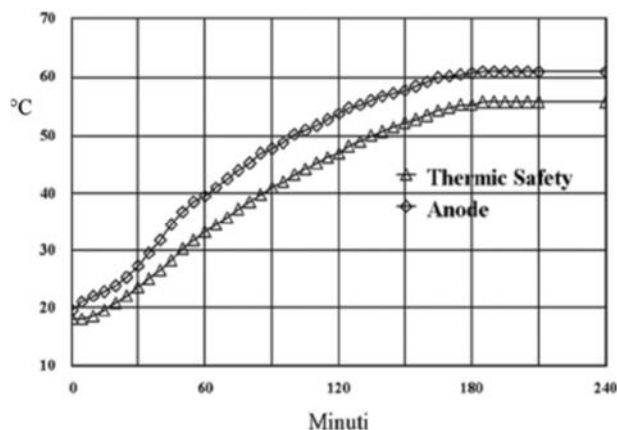
LESS THAN 1 mGy IN ONE HOUR (120 kV-4mA) IN FLUOROSCOPY MODE  
ACCORDING TO IEC 601-1-3 APPLICABLE STANDARD (PAR. 29.204)

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
#### HEATING CURVE (60 kV, 1 mA)

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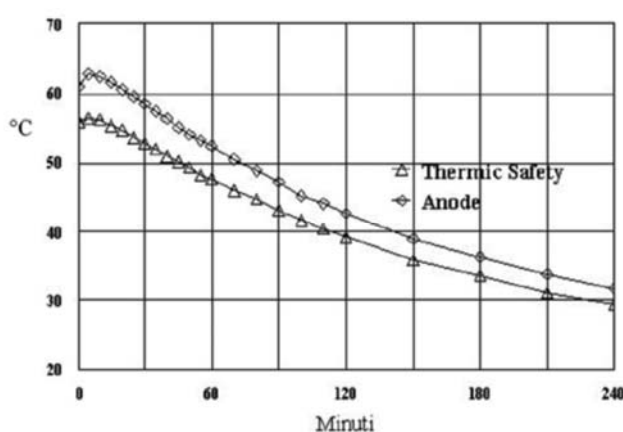#### COOLING CURVE

IMD CHINA CO., LTD. • 1-2/F, BUILDING 12, No. 28 YUHUA RD. AREA B, TIANZHU AIRPORT INDUSTRIAL ZONE, BEIJING, CHINA

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
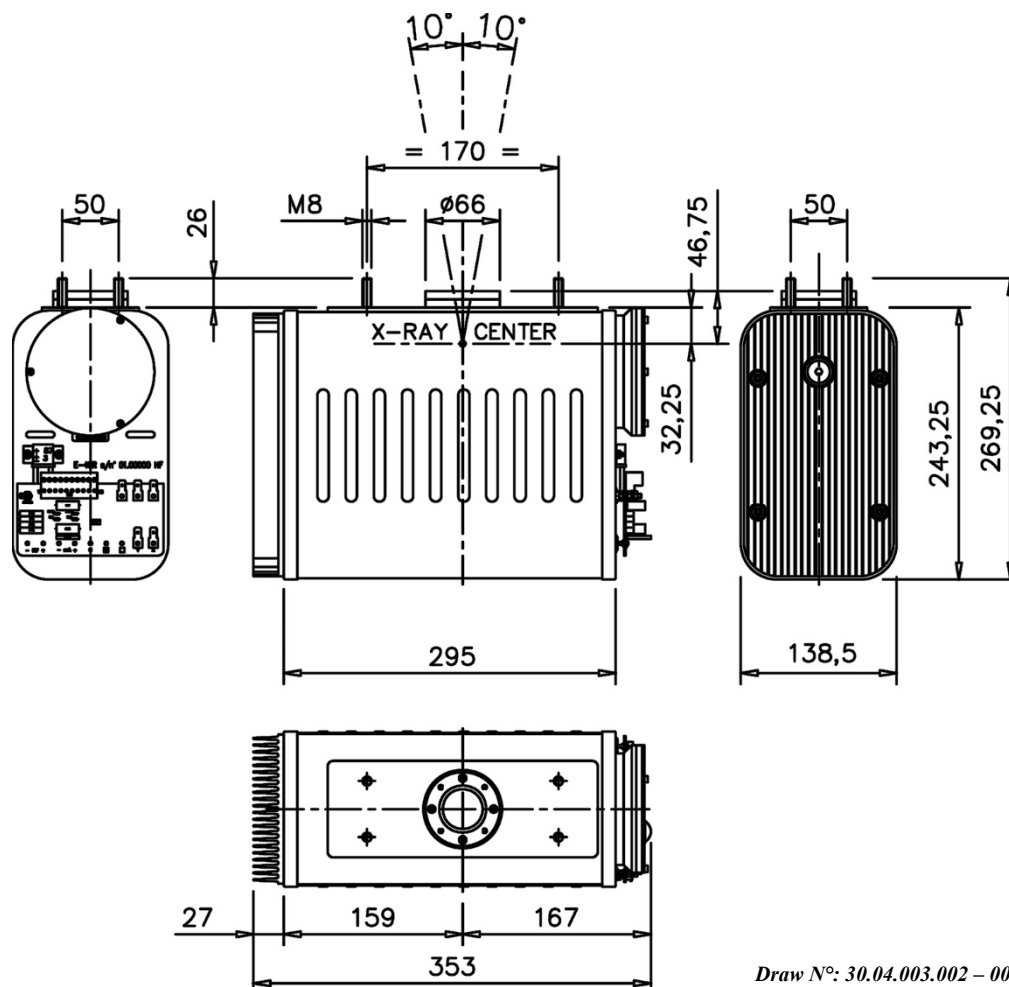
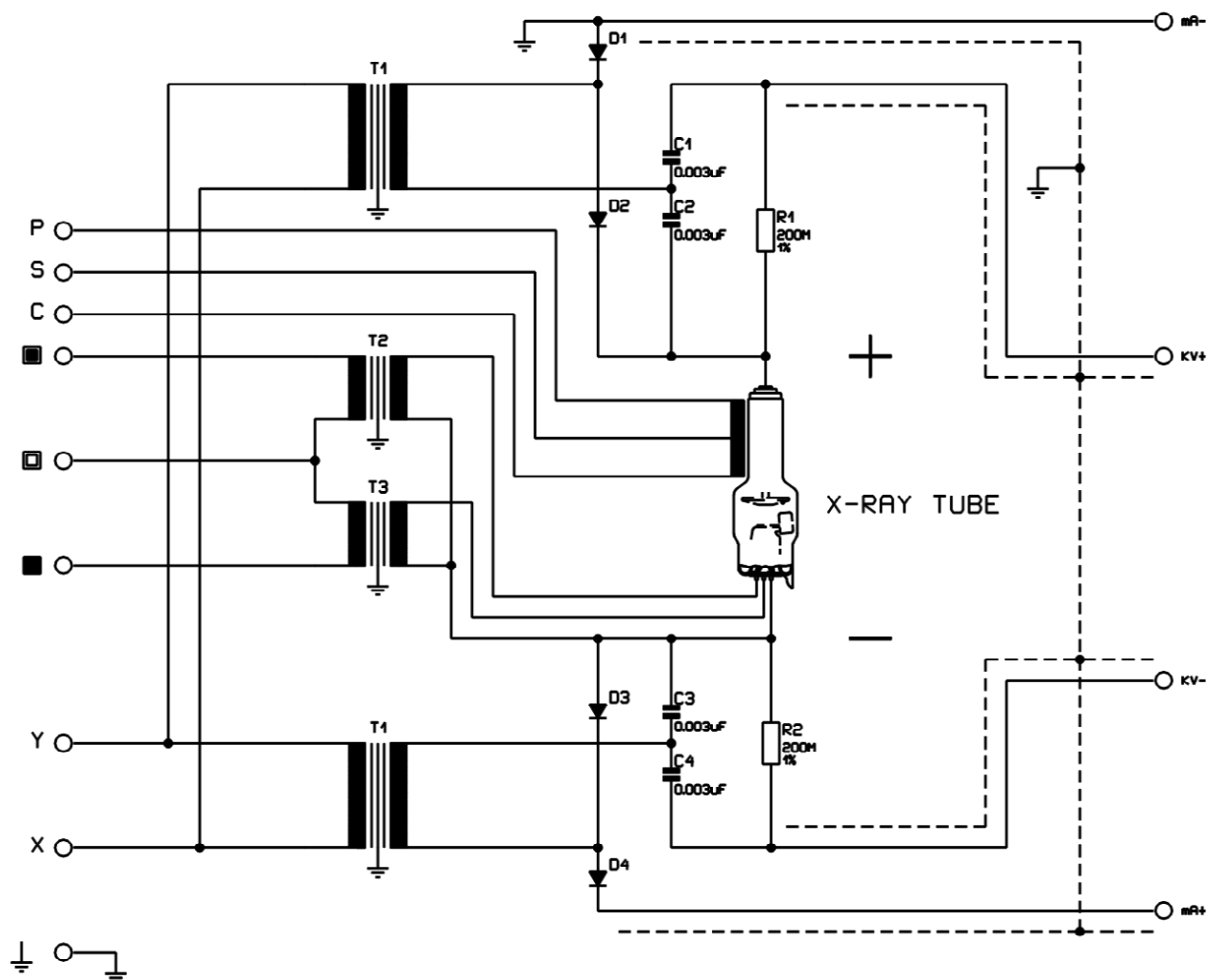
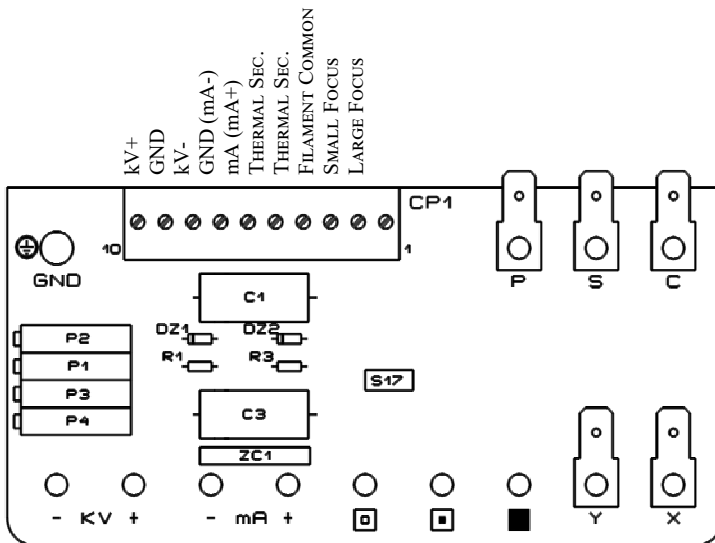
Optionals: 2 disks of aluminium (thickness 1mm), 41/20 o 41/25 lead washer (thickness 2 mm) and seeger.

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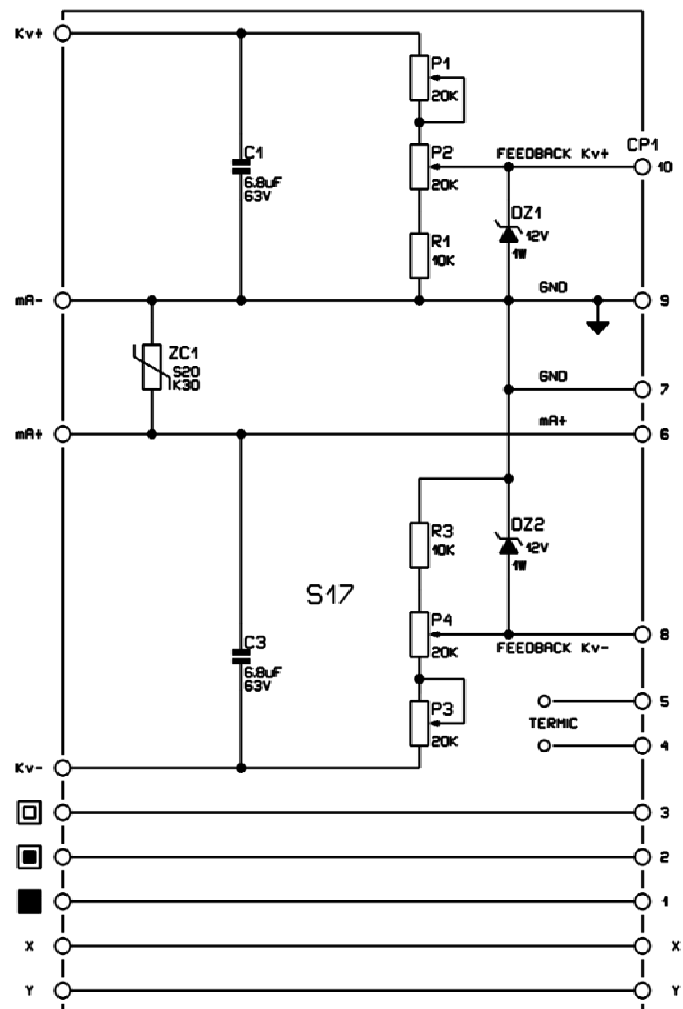# Hazards

PROPER USE AND SAFE OPERATING PRACTICES WITH RESPECT TO X-RAY MONOBLOCK GENERATORS ARE RESPONSIBILITY OF EQUIPMENT MANUFACTURERS AND USERS OF SUCH DEVICES. THE MANUFACTURER PROVIDES INFORMATION ON ITS PRODUCTS AND ASSOCIATED HAZARDS, BUT IT ASSUMES NO RESPONSIBILITY FOR AFTER-SALE OPERATING AND SAFETY PRACTICES. LIMITED LIFE, RANDOM FAILURE AND PERFORMANCE REDUCTION ARE INHERENT CHARACTERISTICS OF X-RAY MONOBLOCK GENERATORS. APPROPRIATE SAFE OPERATING PRACTICES SHOULD BE EMPLOYED.

**ALL PERSON WHO WORK WITH MONOBLOCK OR EQUIPMENT WHICH UTILIZES SUCH DEVICES MUST TAKE PRECAUTIONS TO PROTECT THEMSELVES AGAINST RADIATION EXPOSURE AND POSSIBLE SERIOUS BODILY INJURY.**

**DO NOT OPERATE THIS MONOBLOCK EXCEPT IN ACCORDANCE WITH INFORMATION INCLUDED IN THIS TECHNICAL DATA SHEET, THESE PRECAUTIONS, AND ANY ADDITIONAL INFORMATION PROVIDED BY THE MONOBLOCK MANUFACTURER AND/OR COMPETENT SAFETY AUTHORITIES.**

## X-RAY MONOBLOCK GENERATOR HAZARDS

### HIGH VOLTAGE

All X-Ray Monoblock Generators operate at voltages high enough to kill through electrical shock. Equipment designed should prevent personnel contact with high voltages. When direct access to the Monoblock is required, the primary circuits should be disabled and capacitors discharged.

### X-RAY RADIATION

This Device produces X-Ray radiation that varies in quantity and energy level, depending upon the voltage and current used when energizing it. Refer to the operating instructions for the equipment. Operating personnel must be protected by appropriate shielding. X-Ray Caution signs or labels must be permanently attached to equipment directing operating personnel never to operate this device without X-Ray shielding. Shielding should be in accordance with applicable Norms. If there is any doubt as to the adequacy of shielding, an expert in this field should be contacted to perform an X-Ray survey.

### STORAGE AND CONSERVATION

To guarantee the maintenance of integrity of the product is advised to use or to conserve the system in environments that respect the following climatic factors us.

| Humidity |      |         |      | Temperature |       |         |       | Atmospheric pressure |         |         |         |
|----------|------|---------|------|-------------|-------|---------|-------|----------------------|---------|---------|---------|
| Use      |      | Storage |      | Use         |       | Storage |       | Use                  |         | Storage |         |
| min.     | max. | min.    | max. | min.        | max.  | min.    | max.  | min.                 | max.    | min.    | max.    |
| 30%      | 75%  | 10%     | 90%  | 10 °C       | 40 °C | -20 °C  | 70 °C | 500hPa               | 1600hPa | 500hPa  | 1600hPa |

PRECAUTION SHOULD BE TAKEN IN THE DISPOSAL OF X-RAY MONOBLOCK GENERATORS CONTAINING DIELECTRICAL OIL AND/OR LEAD IN ORDER TO COMPLY WITH ALL APPLICABLE EUROPEAN LAWS AND REGULATIONS. IT IS RECOMMENDED THAT FAILED DEVICES BE RETURNED TO THE MANUFACTURER OR AN APPROPRIATE FACILITY TO ENSURE PROPER HANDLING.

### GLASS EXPLOSION

All X-Ray tubes are pumped to very high vacuum; those incorporating glass are subject to accidental breakage. Breakage will result in an implosion followed immediately by an explosion, yielding flying glass fragments. When handling, safety glasses (or face shield), heavy clothing, and leather gloves should be worn for protection.

### HOT OIL

Extreme heat occurs in the anode portion of the X-Ray tube during operation. This heat is radiated to the dielectric oil inside the Monoblock. This heat is then conducted through the Monoblock and radiated to the air. Misuse of the Monoblock can result in a rupture of the housing causing hot oil to escape, potentially resulting in scalds or burns. Take precautions to avoid such rupture or contact.

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